# Prothyl Hot

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SOLO PER USO DENTALE

### PROTHYL HOT

*Heat curing acrylic resin for dentures*

**Colours:** V2 – Orange Pink Veined  
V4 – Purple Pink Veined  
T2 – Orange Pink  
T4 – Purple Pink

#### Composition

Heat curing acrylic resin made up of:

powder: polymethyl methacrylate  
liquid: methyl methacrylate, dimethacrylate

**Indications:** Total prostheses - Partial prostheses

**Warnings:** Resins based on methylmethacrylate may cause irritation if they come into contact with the skin. If these conditions arise, suspend the use of the product.

**Instructions for use**

**Preparing the model:** Before wedging, isolate with Prothyl Isolator (Zhermack spa) and leave to dry. Roughen and brush the surfaces of the teeth that are to come into contact with the resin, with monomer.

**Dosing:** Ideal mixing proportion: 10 gr. of powder / 4.4 ml. of liquid.

To produce a medium size denture (superior or inferior), it is necessary to use one measure of powder (18.5 g) and 8.1 ml of liquid.

**Mixing:** Mix the powder and liquid into a glass mortar and leave in the closed container. The maturation time is about 20 min. at room temperature, after which the mass must be workable without stringing or sticking to the walls of the container. Wearing gloves, work the mass for about one minute and move on to the next phase.

**Packing and pressing:** Put the material in the flask and press progressively at intervals with a pressure of up to 80 – 100 bars for about 10 min.

#### Standard curing

Immerse the flask blocked by the bracket in the curing apparatus at a temperature of 60°C. Increase the temperature up to 100°C in about 30 min. Maintain this temperature for 30 min.

#### Variants

Other curing procedures, normally of the slow or fast type, may be adopted. In the case of slow curing, bear in mind that the content of residual monomer can only be reduced by increasing the temperature. The ideal result is obtained using the standard method indicated. In the case of fast curing, we recommend the use of the specific resin Prothyl Speed (Zhermack spa)

**Cooling:** Leave to cool for at least 30 min. at room temperature, then complete cooling under running water.

#### Removal from flask/ finishing

Once cooled, open the flask, remove the gypsum and carry out the finishing procedure in the ordinary way with a tungsten bur and rubber. For polishing we recommend Prothyl Polisher (Zhermack spa).

#### Repairs and corrections

Repairs or corrections may be carried out using Prothyl Repair (Zhermack spa), taking care to roughen and dampen the surfaces to be treated, with monomer.

**Maximum residual monomer content:** < 1.2 %

#### Storage

Store in a ventilated room protected from the light at a temperature of between 5–27°C out of the reach of children. Adopt a working practice to ensure that the product is not released into the environment.

**Classification in accordance with ISO 1567**

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**Couleurs:** V2 – Orange Pink Veined  
V4 – Purple Pink Veined  
T2 – Orange Pink  
T4 – Purple Pink

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